Datenlogger  
Data Logger  
Enregistreur

Series EBI 12

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## Table of Contents

<b>Table of Contents</b>	<b>3</b>
Safety and warning instructions	5
<b>Overview</b>	<b>7</b>
The data loggers	7
The interface	9
<b>Information for data loggers with flexible probes</b>	<b>11</b>
<b>Information for data loggers with humidity sensor</b>	<b>13</b>
<b>Unpacking/Scope of delivery</b>	<b>15</b>
<b>Putting the data logger into operation</b>	<b>17</b>
Data transfer mode	19
Using the intermediate ring	21
<b>Replacing the battery</b>	<b>23</b>
<b>What to do if...</b>	<b>29</b>
<b>Cleaning and care</b>	<b>33</b>
Calibration service	33
Disposal	33
<b>Technical data</b>	<b>35</b>
General:	35
Depending on logger type:	37
<b>Spare parts:</b>	<b>37</b>
<b>Approvals</b>	<b>39</b>
General:	39
Foodstuff:	39
Wireless:	41

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## Safety and warning instructions

Do not place the data logger in an environment that is hotter than 150 °C.

Do not expose the unit to microwave radiation.

Danger of explosion of the lithium battery!

Do not touch the logger with bare hands immediately after a measurement. Danger of burns!

The data logger must be kept in watertight packaging for immersion in liquids below -20 °C.

**Only use the batteries available from *-ebro-*<sup>®</sup> for these data loggers. Only with these batteries the logger fulfills the requirements of the EX approval and only these have the required temperature resistance!**

The data loggers transmit radio signals in the frequency range of 2.4 GHz.

This can significantly disrupt other technical equipment under certain conditions. Only use the data loggers in areas where the radio waves pose no danger.

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## Overview

### *The data loggers*

The EBI 12 series of data loggers are high-precision measuring and recording units for temperature, humidity and/or pressure. They are operated with a lithium battery, and programmed and read using a PC.

A suitable interface (available as an accessory) and corresponding analysis software on the PC (e.g. "Winlog.validation", available as an accessory) are required for programming the data logger. The interface is connected to the PC via Universal Serial Bus (USB).

The EBI 12 series of data loggers are available in various versions for a range of applications.

- Channels 1, 2, 3 or 4
- Temperature, temperature/humidity or temperature/pressure
- Temperature sensor versions, flexible or rigid  
Ø 1.5, 3.0 or 6.0 mm  
Lengths from 10 to 500 mm  
Radial or axial position  
Pointed or blunt version
- Cable probe Ø 1.2 mm  
Lengths of 600 mm and 1200 mm
- Pressure sensor connection  
Standard  
M10 internal thread for hose, flange, or glass adapters  
Luer lock

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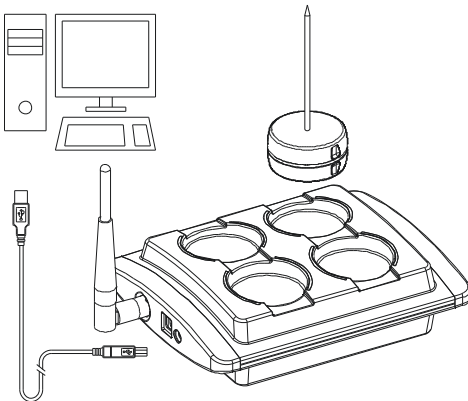
For programming the data logger and for reading the recorded data, they are placed in the interface.

For more information on programming the data logger and for reading the data, please consult the user manual of your analysis software.

The data loggers do not have an on/off switch. They are in operation as soon as (and for as long as) the battery provides power.

### *The interface*

A suitable interface (available as an accessory) is required for programming the data logger. This interface is connected to the PC via a Universal Serial Bus (USB, available as an accessory) and powered from there or by its own power supply (available as an accessory).

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## Information for data loggers with flexible probes

Do not bend the probe in the first 10 mm at the housing nor in the 50 mm at the probe tip.

This can damage the probe. If bent close to the housing, the mechanical load is too great due to the edge of the housing (figure 1).

The sensor is located in the 50 mm at the tip of the probe.

Do not use pliers to bend it. When bending, hold the probe tight in the lower area with two fingers and bend it into the required position with the other hand.

The bend radius must be at least 10 mm. Smaller bend radii lead to material fatigue and hence to breakage of the probe.

Leave the data logger (which is in a silicone protective box) in the protective box. The pulling and threading of the flexible probe can subject it to undue stress and possibly damage it.

To read or program the data logger (in a silicone protective box), pull off only the bottom part of the protective box downwards.

After reading or programming the data logger, close the silicone protective box again.

The cable probes can be exchanged individually in the factory.

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## Information for data loggers with cable probes

Do not bend the probe in the first 10 mm at the housing nor in the 50 mm at the probe tip. This can damage the probe.

When bending close to the housing, the mechanical load is too great due to the edge of the housing. The sensor is located in the 50 mm at the tip of the probe.

Do not use pliers to bend it. When bending, hold the probe tight with two fingers in the lower area and bend it into the required position with the other hand.

The bend radius must be at least 10 millimetres. Smaller bend radii lead to material fatigue and hence to breakage of the probe.

Do not guide the cable over edges.

Do not pull on the cable.

## Information for data loggers with humidity sensor

**Never open the sinter cap in an area with a potentially explosive atmosphere.**

For instructions on changing the humidity sensor, please refer to the instructions included with the replacement sensor.

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## Unpacking/Scope of delivery

Upon receipt, inspect the unit packaging and check that the contents are intact and complete.

The scope of delivery includes the following parts:

- EBI 12 data logger
- This user manual
- Calibration service card

In addition, the delivery can also include various accessory parts:

Description	Designation
Programming interface	EBI IF100/200
USB cable	
Programming software	Winlog.x
Antenna	AL 111/112
Silicone protective box	AL 100/101

Spare parts that can accompany the delivery:

Description	Designation
Battery change set	AL 120/104

**Only with these batteries the logger fulfills the requirements of the EX approval and only these have the required temperature resistance!**

If there are any reasons for complaint, please get in touch. Our contact data can be found on the back of the envelope.

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## Putting the data logger into operation

The data loggers do not have an on/off switch. They are in operation as soon as, and as long as, the battery provides power.

However, they must be programmed for their use. This is performed on a PC or Laptop to which a special read/write device (interface) is connected using special software (e.g. "Winlog.validation").

To program an EBI 12 data logger, put it into one of the interface ports so that the side with the plastic insert points to the interface. Then carry out the programming with the aid of the programming software.

Further details of this can be found in the user manual for the corresponding software.

Always allow the data logger to cool down slowly from high temperatures to a low temperature. Rapid cooling can impair the leak-tightness of the data logger.

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## *Data transfer mode*

### Wireless mode:

The logger sends its data to the interface from a distance at a programmed interval.

### Interface mode:

The logger is placed in the interface for programming and reading the data.

If the measurements are to be transferred, make sure there is permanent wireless connection between the data logger and the interface (for further details of this, see the manual for the interface used) and that the analysis program on the computer can also continuously receive the incoming data (for further details of this, see the manual for the corresponding software).

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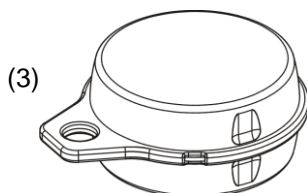
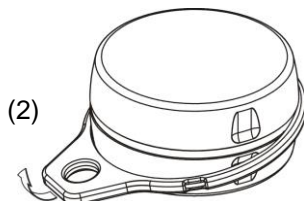
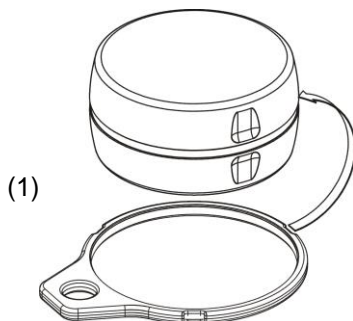
*Using the intermediate ring*

To insert the eyelet ring, it is not necessary to open the housing.

As standard, the EBI 12 series data loggers do not have an eyelet.

An intermediate ring with eyelet can be used if required.

A suitable intermediate ring is available as an accessory.

To install the intermediate ring, proceed as follows:

- Put the eyelet ring into the Groove provided on the logger (Figure 1).
- Now slide the eyelet ring over the logger (Figure 2).
- Make sure that the eye ring seated securely into the Groove provided for this purpose (fig. 3).

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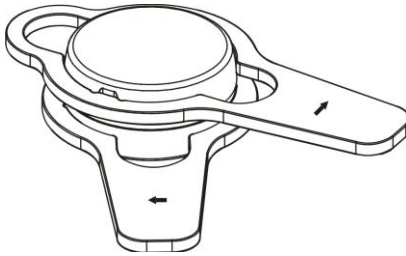
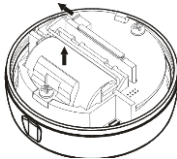
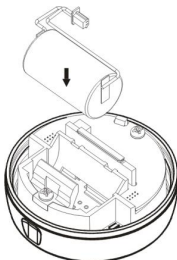
## Replacing the battery

Once a battery is spent, it must be replaced with another one.

**Only use the batteries available from **-ebro-®** for these data loggers. Only with these batteries the logger fulfills the requirements of the EX approval and only these have the required temperature resistance!**

**Whilst changing the battery they must protect themselves against electrostatic discharges using suitable precautionary measures such as an earthed wristband! Electrostatic discharges may destroy the device!**

To swap out a battery:

- Use the opening tool to open the housing of the data logger (see Figure 1).
- Pull the battery plug out of the circuit board (Figure 2).
- Remove the empty battery.

**Dispose of the used battery in an environmentally friendly way at a suitable re-cycling point!**

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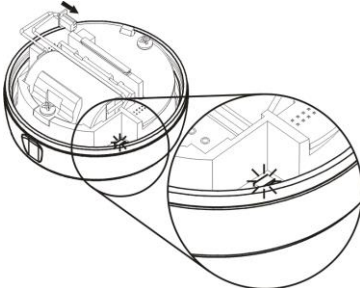
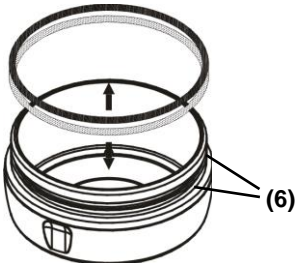
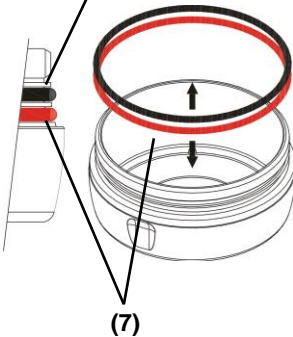
- Place a new battery in the battery compartment in exactly the same position as the one you have just removed (Figure 3).
- Plug the battery plug carefully into the socket on the circuit board (Figure 3).

**Take great care throughout this procedure and do not use excessive force at any point. The plug is polarized and can only fit into the socket on the circuit board in one direction.**

To the right of the battery compartment, you will see a LED. This LED will flash as soon as the circuit board is receiving power from the battery again (Figure 4).

Remove the elastic O-rings from the bottom part of the logger housing (Figure 5).

- Cleaning the slots in which the O-rings were located (Figure 6).
- Take new O-rings, cover it with a thin layer of grease and place it in the slots (red = Figure 7, black = Figure 8).

**The layer of grease should cover the O-rings completely (so that it glistens) but not leave lumps of residue on it.**

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**Take care not to overstretch the elastic ring and not to damage it!**

- Screw the two parts of the data logger back together again, initially by hand to be hand-tight.
- Then please use the opening tool for further finish until no more visible gap.
- Wipe away any grease that may be squeezed out.
- Use the programming software to reset the battery counter.

For more information, please refer to your programming software's operating instructions.

**No data is lost when you change a battery or if the battery fails. However, if a longer power failure occurs, the built-in clock will no longer show the correct time. In this situation, extract the data from the data logger and then reprogram it. This resynchronizes the date and time.**

**Recording continues immediately after you insert a new battery.**

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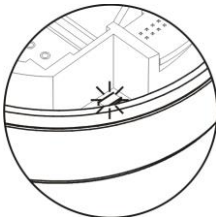
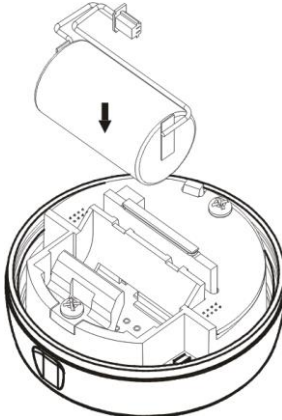
**What to do if...**

- ... there is no wireless communication with the data logger:
- Check that the wireless mode of the data logger is active. Place the data logger close to the interface and call up the corresponding function of the programming software.
  - If necessary, switch the data logger to wireless mode with the aid of the programming software.
  - If the data logger does not respond, check if the battery is flat and needs replacing (see section "Replacing the battery").
- ... the data logger is not responsive in the interface:
- Check that the interface is correctly connected and has been detected by the analysis software.
  - Open the data logger (see the section "Replacing the battery").
  - Check if there is moisture in the data logger housing.
  - Remove the moisture from the housing or allow the open data logger to dry sufficiently in the air.

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- Even if there was no moisture in the data logger housing: Remove the battery plug from the socket and wait 30 seconds.
- Then insert the battery again and check if the LED to the right of the battery compartment flashes. If the LED does not flash, the battery may be depleted and must be replaced (see the section "Replacing the battery".)
- To check, insert a new battery in the socket on the PCB. If the LED still does not flash with the new battery, the data logger is obviously faulty.
- Before closing the data logger housing again, always replace the O-rings.

**A used O-ring must always be disposed of. Under no circumstances should it be reused! This is because it is deformed when the two parts of the housing are screwed together. If it is used again it will not make a proper seal!**

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## Cleaning and care

Only clean the outside of the data logger with a slightly moist cloth.

**Do not use solvents such as acetone.**

### *Calibration service*

The unit has high measuring accuracy. To maintain this, the unit should be calibrated annually.

We offer a calibration service for this purpose.

To request this service, please complete the accompanying service card and return it to us.

We will then call in your unit for calibration on an annual basis.

The calibrated unit will be sent back as soon as possible.

### *Disposal*

If the unit becomes unusable, it must be disposed of properly and in an environmentally-responsible manner.

**Never dispose of the unit with household waste; return it to the manufacturer.**

**Dispose of the battery at the collection points provided.**

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### General:

#### Measurement variables

Pressure	mbar (kPa),	1 pressure channel
Humidity	% rH,	1 humidity channel
Temperature	(°C),	1-4 temperature channels
Conductivity	µS/cm	1 conductivity channel

#### Measuring accuracy

Pressure:	±10 mbar
Low pressure: (TLP)	±0.25 mbar
Humidity:	±2% (10 to 90% and 25 °C)
Temperature:	±0.05 °C (120 to +140 °C) ±0.1 °C (0 to +120 °C) ±0.2 °C (-40 to 0 °C) and (+140 to +250 °C) ±0.5 °C (-90 to -40 °C) and (+250 to +400 °C) ±1.5 °C (-200 to -90 °C)
Conductivity	±0,5µS/cm

#### Resolution of the measurement signal

Pressure:	1 mbar (100 Pa)
Low pressure: (TLP)	0,1 mbar (10 Pa)
Humidity:	0.1%
Temperature:	0.01 °C
Conductivity	0,1µS/cm

#### Data memory

up to 100,000 measurements

#### Sensors:

Pt 1000  
piezo-resistive pressure sensor  
capacitive humidity sensor  
capacitive conductivity sensor

#### Interface/Wireless frequency:

Wireless 2.4 GHz / IEEE 802.15.4

#### Reliable operating temperature of the data logger

Log mode:	-90 °C to +150 °C
Wireless operation:	-30 °C to +150 °C

#### Storage

EBI 12-T cable probe	-20 °C to +70 °C
(otherwise)	-40 °C to +125 °C)

#### Protection class:

IP68

#### Battery:

Lithium battery 3.6 V, replaceable

Battery life up to 2 years depending on the application

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AL104 (3 Batterien) 1100-0118

*Depending on logger type:***Measuring range**

<b>EBI 12-T series type 100</b>	-90 to +150 °C
<b>EBI 12-T series type 101</b>	-90 to +85 °C
<b>EBI 12-T series, Ø probe 3 and 6mm</b>	-40 to +150°C
<b>EBI 12-T series, Ø probe 1,5mm axial</b>	-200 to +200°C
<b>EBI 12-T series, Ø probe 1,5mm radial</b>	
Length = 250mm	-200 to +200°C
Length = 500mm	-200 to +400°C
<b>EBI 12-T series cable probe</b>	-20 to +150°C
<b>EBI 12-TP series</b>	
Pressure:	0 to 4000 mbar (0 to 400 kPa)
Temperature:	0 to +150 °C
<b>EBI 12-TLP series</b>	
Pressure:	0 to 1050 mbar (0 to 105 kPa)
Temperature:	0 to +85 °C
<b>EBI 12-TH series</b>	
Humidity:	0 to 100%
Temperature:	-40 to + 85°C
<b>EBI 12-TC series</b>	
Conductivity:	0 to 2000µS/cm
Temperature:	0 to + 125°C

**Spare parts:***Battery replacement set*

[Battery type: TLH-5902 or SL-550]

[Manufacturer: Tadiran Batteries]

AL120 (3 batteries incl. changing tool)	1100-0130
AL104 (3 batteries)	1100-0118

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## Approvals

### *General:*

This product fulfils the Electromagnetic Compatibility of Equipment Act (EMVG) according to the certificate of conformity.

### *Foodstuff:*

The EBI 12 product fulfils the directives in accordance with the standard EN 12830.

Suitability:

S (Storage), T (Transport)

Environment: A, B, C, D

Accuracy class:

1, for measuring the air temperature and temperature inside the product.

Measurement range: -40 ... +150°C

Please note that this unit (as per EN 12830) must undergo a regular check in accordance with EN 13486 (annual).

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**Wireless:**

This unit has been tested according to ETSI EN

It fulfils the requirements of ETSI EN:

EN 61326-1:	2006
EN 62479:	2010
EN 301 489-1 V1.9.2:	2011
EN 301 489-17 V2.2.1:	2012
EN 300 220-1 V2.4.1:	05/2012
EN 300 220-2 V2.4.1:	05/2012
ETSI EN 300 328 V1.9.1:	02/2015
ETSI EN 300 330 V1.7.1:	02/2010

This unit has been tested in accordance with CFR 47 Part 15 FCC Rules and Regulations CFR 47 Part 15 Subparts B and C dated May 2007

It fulfils the requirements of Part 15, FCC Rules.

This unit has been tested in accordance with RSS Rules, RSS Rules and Regulations

RSS-Gen Issue 2, RSS-102 Issue 2, RSS-210 Issue 7

This unit fulfils the requirements of the RSS Rules.

Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) This device must accept any interference received including interference that may cause undesired operation.

This device has been designed to operate with an internal antenna.

Other Antennas are strictly prohibited for use with this device.

Modifications of any type to the unit will invalidate the operating permit.

Information, operating manuals and declarations of conformity can be found at [www.ebro.com](http://www.ebro.com).

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Xylem Analytics Germany Sales GmbH & Co. KG
Xylem Analytics Germany GmbH